

eLight

Skin Tightening/
Wrinkle Reduction

Skin Rejuvenation

Acne



*Complete Facial
Aesthetic Solution*

eLight™

Syneron™

CLEAR, TIGHTEN & REJUVENATE SKIN.

eLight is a complete facial treatment system. Proven safety and efficacy for a wider range of skin textures and tones means eLight



can help build your practice.

THE eLōs TECHNOLOGY ADVANTAGE.

The eLight features proprietary eLōs combined-energy technology of Bi-Polar Radio Frequency (RF) with light energy.

- Increased selectivity, deeper dermal penetration
- Less optical energy, when synergistically combined with Bi-Polar RF, improves the safety factor when treating all skin types

How eLōs works.



Fig.1



Fig.2



Fig.3

Fig.1 Light pre-heats target.

Fig.2 RF is drawn toward heated target deep inside the dermis.

Fig.3 Both energies work to attack both the inside and outside of the target.

ENGINEERED for SAFETY.

- Proprietary Active Dermal Monitoring™ feature monitors dermal temperature changes during each pulse for added safety
- Contact cooling promotes patient comfort
- No patient downtime

EASY to USE & MAINTAIN.

- Complete your own upgrades. No tools, no technicians
- Portable, lightweight, small footprint





Skin Tightening/ Wrinkle Reduction



Photos: James Bartholomeusz

Skin Rejuvenation



Photos: Amy Taub, M.D.

Acne



Photos: Lisa M. Kellett, M.D.

FULL FACIAL APPLICATIONS.

ReFirme™ Skin Tightening [ST Applicator]

The ST Applicator firms, tones and tightens lax skin via wrinkle treatment with the eLōs combination of infrared optical energy and Bi-Polar Radio Frequency. Achieve exceptional, immediate patient satisfaction with enhanced comfort.

Skin Rejuvenation/FotoFacial RF™ [SR and SRA Applicators] The SR Applicator rejuvenates the appearance of skin by treating superficial benign vascular and pigmented lesions including telangiectasias, symptoms

pigmented lesions, but not significantly absorbed by the normal dermis. As featured in this eLight system application, eLōs is the exclusive technology of the FotoFacial RF procedure.



Choose the SRA Applicator to aggressively treat stubborn and resistant superficial benign pigmented and vascular lesions on lighter skin colors.

Acne Treatment [AC Applicator] Reduce sebaceous gland activity and destroy active acne. The combined energies of eLōs – natural photodynamic therapy (400-980-nm spectrum) simultaneously combined with RF – enables highly selective targeting of the sebaceous glands and acne bacteria.

“Patients I have treated using the ReFirme procedure have smoother, plumper skin and see results more quickly than other skin tightening treatments I have used.”

*~ Robert Sleightholm, M.D.
Plastic Surgeon*

of rosacea and angioma. Highly selective broad spectrum light energies are absorbed by the hemoglobin and

eLight SYSTEM SPECIFICATIONS



Application Heads	ST – Skin Tightening SR – Skin Rejuvenation SRA – Advanced Skin Rejuvenation AC – Acne Treatment
RF Energy	
ST	Up to 120 J/cm ³
SR, SRA	Up to 25 J/cm ³
AC	Up to 20 J/cm ³
Light Fluence	
ST	Up to 10 W/cm ²
SR, SRA	Up to 45 J/cm ²
AC	Up to 18 J/cm ²
Spectrum	
ST	700 – 2000 nm
SR	580 – 980 nm
SRA	470 – 980 nm
AC	400 – 980 nm
Cooling	5°C on skin surface
Skin impedance control	Online
Treated area	
ST	12 x 8 mm
SR, SRA, AC	12 x 25 mm
Pulse repetition rate	
ST	1.2 Hz
SR, SRA, AC	0.7 Hz
Platform Weight	55 lbs (25 kg)
Platform Size	35.5 x 15 x 15" (90 x 38 x 38 cm)
Electrical Requirements	Standard 110 VAC (230 VAC)

Syneron has the Proof that matters most to your Success.

Proven Technology.

Proven In-Demand Applications.

Proven, Documented Clinical Efficacy.

Proven Smart Investment.

View our growing list of clinical papers and treatment photos.

Get detailed product specifications. Learn how to optimize your practice's success with eLight.

Visit us online at www.syneron.com.

Syneron Inc.
111 Granton Drive, Unit 110
Richmond Hill, Ontario,
Canada L4B 1L5
Tel: 905-886-9235
Toll Free: 1-866-259-6661

Syneron Medical Ltd.
Industrial Zone, Tavor Building
P.O.B. 550 Yokneam Illit,
20692 Israel
Tel: +972 4 909-6200

Syneron™